

WEST PALM BEACH FIRE DEPARTMENT



DECON SPRAYER/ FOGGER



PURPOSE: DECON OF PATIENT COMPARTMENT AFTER MED ALERTS

- High touch areas
- Stretcher

EQUIPMENT

- One (1) Wagner sprayer 350.
- One (1) HDX 25 ft Indoor/Outdoor Extension Cord.

PICK ONE (1) OF THE FOLLOWING CLEANING SOLUTIONS

- Cavicide (no dilution required). Allow 3 minutes to sit on surface.
- 70% Isopropyl Alcohol (no dilution required). Allow 3 minutes to sit on surface.
- Bleach (Mix 2 caps bleach per full container of water). Allow 5 minutes to sit on surface.

**DO NOT MIX THE ABOVE PRODUCTS TOGETHER IN THE
SPRAYER!! FLUSH SPARYER WITH H2O PRIOR TO ADDING A
DIFFERENT SOLUTION!!**

KEY COMPONENTS



ON/OFF



**FLOW CONTROL:
SET TO #2**



**AIR CAP: ADJUST FOR
HORIZONTAL OR VERTICAL
SPRAY**



**SPRAY WIDTH LEVER:
SET TO NARROW FOG**

PROCEDURE

1. DO NOT USE ON ELECTRONIC EQUIPMENT TO INCLUDE MONITORS AND RADIOS!
2. Plug sprayer into outlet.
3. Add recommended cleaning solution to container.
4. Set “Flow Control” to #2 (as shown above).
5. Set “Spray Width Lever” to narrow fog position (as shown above).
6. Adjust the “Air Cap” position for Horizontal or Vertical pattern (as shown above).
7. Spray all surfaces, equipment, and high touch areas.
8. Open all windows to allow complete ventilation while returning to station.
9. WEAR ALL APPROPRIATE PPE DURING PROCEDURE (i.e. eye protection, N95 mask, gloves).
10. SPRAYER OPERATOR DISCARDS N95.

SEE OWNER’S MANUAL FOR CLEANING SPRAYER OR ANY OTHER FREQUENTLY ASKED QUESTIONS.

VIDEO SHOWING SET UP OF THE SPRAYER, VISIT:

T.S. >FILE CENTER>COVID-19>COVID-19 LINKS